

POWER ATRAUMATIC FORCEPS

Reorder #5040



CONTENTS

1 Universal F1P Upper Premolar
1 Universal F2P Lower Premolar

1 Universal F3P Upper Anterior
1 Universal F4P Lower Anterior

1 Universal F6P Upper Molar
1 Universal F7CP Lower Molar

USE

To assist in grasping and removing teeth or root fragments during dental procedures.

PRIOR TO USE

- Inspect the instrument for any sign of damage, wear, or defects before each use.
- Sterilize before each use. Steam sterilize at 270°F/132°C for 4 minutes or 250°F/121°C for 30 minutes.
- **Recommended:** Allow 30 minutes of dry time after the sterilization cycle.
- Do not heat above 320°F/160°C. Use recommended chemicals only.
- Use only manufacturer-approved cleaning agents.
- Detailed care and maintenance instructions available upon request or go to powerdentalgroup.com.

CAUTION

- Minimal hand grip pressure may be required when using these forceps.
- Avoid excessive pressure to prevent tooth fractures.
- Ensure proper forceps selection for the specific tooth.
- Avoid pinching soft tissue during extraction to minimize trauma.

DIRECTIONS

STEP 1

Selection: Choose the appropriate forceps based on tooth location and clinical need.

STEP 2

Grip and Position: Grasp the tooth or root securely at a point that minimizes trauma to surrounding tissues.

STEP 3

Apply Force: Use controlled force based on the extraction technique required for the clinical scenario.

STEP 4

Extract: Elevate and remove the tooth or root while minimizing damage to adjacent structures.

MAINTENANCE

- Inspect instruments after cleaning and replace if damaged or worn.
- Lubricate hinges as needed.
- For optimal precision return to Power Dental for sharpening if the working edge becomes dull.



F1P Upper Premolar



F2P Lower Premolar



F3P Upper Anterior



F4P Lower Anterior



F6P Upper Molar



F7CP Lower Molar