



HEALING ABUTMENT FORCEPS

Gently holds and stabilizes healing abutments during the approach, positioning and placement into the implant. The driver easily screws the healing abutment into the implant while being secured by the forceps.



HAF/90 - The perfect universal forceps.


*The prongs of the forceps are together when the instrument is at rest.**

* The forceps will allow conventional finger twisting of the driver while holding, stabilizing and screwing in the healing abutment.



HAF/75 - Offset - Allows line of sight advantage. Perfect for use with mirror or microscope. Ideal for placement in molar areas.

METHOD OF USE:

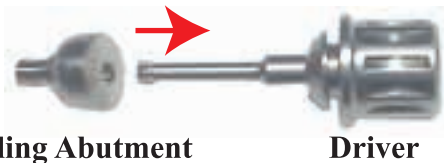

Healing Abutment

Driver



Attach driver to healing abutment



Spread the handles of the forceps apart, place the healing abutment between the semilunar prongs and allow the prongs to gently come to rest on the abutment.



Use the forceps to approach the implant, place into position and screw into place.