


neo **Remover** *Screw* kit

**An alternative solution
to remove
fractured screws**

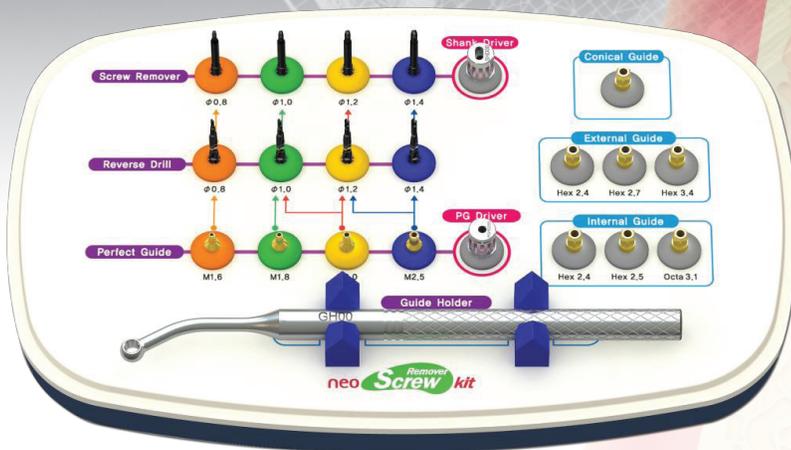

Simple

- Simple removal of fractured screw
- Can be used for most implant systems


Easy

- All in one instrumentation

neo **Screw Remover** kit

Screw Remover Kit (SRKIT) includes one (1) of each component listed.

Instrument kit for removing fractured screws from the implant

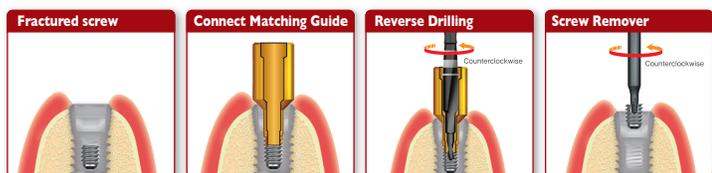

Perfect Guide is used if the fractured screw is less than 2mm within the implant female chamber.
* For greater than 2mm options to use internal and external guides are available.

Speed : 1,200~1,300RPM

Speed : below 80RPM or manually



* Perfect Guide		Application fixture female screw	Application Reverse Drill
Diameter(Φ)	product Name		
Φ0.8/M1.6	PG 0816	M1.6	RCD 08
Φ1.0/M1.8	PG 1016	M1.8	RCD 10
Φ1.2/M2.0	PG 1220	M2.0	RCD 10, 12
Φ1.4/M2.5	PG 1425	M2.5	RCD 12, 14


* Be sure to use the same size of Perfect Guide as fixture female screw..



* PG Driver * Used to connect the Perfect Guide

Hex(mm)	product Name	Application Product
2.5	PGHD 25S	PG0816, PG1018, PG1220, PG1425

Reverse Drill * Used with irrigation

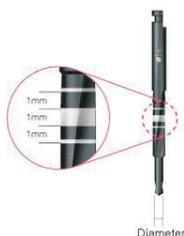

The Reverse Drill is used to mark the sight hole for the fractured screw to receive the screw remover.
The recommended drill speed is 1,200~1,300rpm.

Diameter(Φ)	product Name	Fractured Screw size
Φ0.8	RCD 08	M1.6
Φ1.0	RCD 10	M1.8 or M2.0
Φ1.2	RCD 12	M2.0 or M2.5
Φ1.4	RCD 14	M2.5

Note: This guide is for educational use only.
See instructions for use for additional information on this product.

Conical, Internal & External Guide

Used whenever a perfect guide cannot be used or when a screw is fractured deeper than 2mm within the implant female chamber.

* Conical Guide		
Size	product Name	Application Fixture
11° / 8°	CG 00	11° or 8° internal connection Fixture

* Internal Guide		
Size	product Name	Application Fixture
2.4 Hex	IHG 24	2.4 hex external connection Fixture
2.5 Hex	IHG 25	2.5 hex external connection Fixture
3.1 Octa	IOG 31	3.1 Octa external connection Fixture

* External Guide		
Size	product Name	Application Fixture
2.4 Hex	EHG 24	2.4 hex external connection Fixture
2.7 Hex	EHG 27	2.7 hex external connection Fixture
3.4 Hex	EHG 34	3.4 hex external connection Fixture

Guide Holder



The Guide Holder is used to provide better control and handling of all Guides except for the Perfect Guide.

Standard	product Name	Application Fixture
		Conical Guide : CG00
3.5 Double Hex	GH 00	3.5 Double Hex GH 00 Internal Guide : IHG24, IHG25, IOG31 External Guide : EHG24, EHG27, EHG34

Screw Remover



The fractured screw is retrieved by using reverse rotation with the Screw Remover with less than 80rpm.

Diameter(Φ)	product Name	Application
0.8~1.1	SR 08	In conjunction after Reverse Drill Φ0.8(RCD08)
1.0~1.3	SR 10	In conjunction after Reverse Drill Φ0.8(RCD08)
1.2~1.5	SR 12	In conjunction after Reverse Drill Φ0.8(RCD08)
1.4~1.7	SR 14	In conjunction after Reverse Drill Φ0.8(RCD08)

Shank Driver



The Shank Driver is used to secure the Screw Remover for manual application or if determined used in conjunction with the torque wrench or ratchet.

product Name	Application Product
SHD 00	SR08, SR10, SR12, SR14

Visit us online at www.hubermed.com
or call I (888) 789.9928

Ordering information

Catalog Number Description

SRKIT Screw Remover kit

HUBERMED is the exclusive Canadian distributor for Neobiotech Neo SR kit (Neo Screw Remover Kit)

© 2013 HUBERMED INC. All rights reserved.
Neo Scre Removerkit is manufactured by Neobiotech Co. Ltd. and distributed by HUBERMED INC. in Canada.

HUBERMED

HUBERMED inc.

9900, boul. Cavendish, suite 400,
St-Laurent, Qc., Canada H4M 2V2
Tel.: I (888) 789.9928

Skype: hubermed • E-mail: info@hubermed.com
www.hubermed.com